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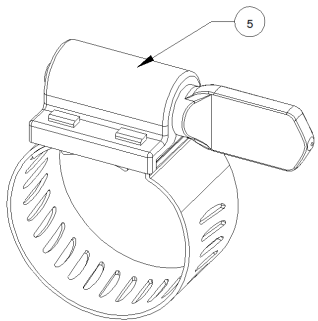
FIGURE 3/FIGURA 3

FIGURE 4/FIGURA 4

INTENDED PURPOSE/INDICATIONS FOR USE

The intended purpose of the ACM Exhaust Hose Connector is an optional accessory of the AtriCure cryolCE BOX, providing a method to connect the AtriCure cryolCE BOX exhaust to a hospital medical vacuum or waste anesthesia gas disposal (WAGD) system. It is intended only to be used together with the AtriCure cryolCE BOX to enable meeting its intended purpose.

WARNING:
Please refer to the CryolCE Box (ACM) for Console Warnings, Cautions, product description, flow rates, and features. Care should be exercised in users with suspected or known allergies or hypersensitivity to nickel.

ACM EXHAUST HOSE CONNECTOR PERFORMANCE CHARACTERISTICS

The ACM Exhaust Hose Connector provides a method to connect the cryolCE BOX exhaust to a hospital Medical Vacuum or Waste Anesthesia Gas Disposal (WAGD) system.

ACM EXHAUST HOSE CONNECTOR CONFIGURATIONS

Part Number	Description	Region
A001150-15	Medical Vacuum Connector Chemetron by ¼" MNPT - Brazil	BR
A001150-16	WAGD Connector Chemetron by ¼" MNPT - Brazil	BR

Item Number	Description
1	Tee Fitting
2	Barbed Fitting
3	Pressure Relief Valve
4	Hospital Specific Connection
5	Hose Clamps

ACM EXHAUST HOSE CONNECTOR INSTALLATION

- Place two hose clamps (item 5) over the clear exhaust tubing.
- Push one end of the clear tubing over the barbed fitting on the back of the cryolCE BOX.
- Push the other end of the clear tubing over the barbed fitting (item 2) on the ACM Exhaust Hose Connector.
- Tighten the two hose clamps and verify both ends of the AtriCure cryolCE BOX clear exhaust tubing are securely attached. This confirms the exhaust hose connector is properly installed.

CAUTION: The exhaust hose can come off the barbs during the freeze cycle if the hose is not secured properly. Connect the hospital specific connection (item 4) to a dedicated Medical Vacuum or WAGD Port.

WARNING:
The ACM Exhaust Hose Connector requires a dedicated Vacuum or WAGD Port to prevent back pressure into the patient's breathing line, which may result in a Pneumothorax.

Preventative Maintenance: No preventative maintenance required.

DISPOSAL OF RE-USABLE DEVICES:

- Disconnect the exhaust hose connector and treat as regulated medical waste requiring decontamination to render safe for further handling and disposal.
- Follow cleaning and disinfecting steps for the exhaust hose connector, as outlined in section 7 of the AtriCure cryolCE BOX IFU.
- Contact local medical equipment recycling and disposal service.

DISCLAIMER

Under no circumstances will AtriCure, Inc. be responsible for any incidental, special or consequential loss, damage, or expense, which is the result of the deliberate misuse of this product, including any loss, damage, or expense which is related to personal injury or damage to property.

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