AtriCure®

Rx ONLY

ASCS (450 mm) Cart

MD

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INSTRUCTIONS FOR USE

en

Information in this document is subject to change without notice. Reproduction of this document in any manner whatsoever, without the written permission of AtriCure® Inc. is strictly forbidden. AtriCure® and the AtriCure® Logo are trademarks of AtriCure®, Inc. AtriCure® disclaims proprietary interest in the marks and names of others.

ASCS (450 mm) Cart

The AtriCure ASCS Cart provides support and mobility for any combination of AtriCure's ACM, ASU, and ASB technologies. This cart is intended for use only in regions that use tank sizes ≤ 20 cm (8 inches) in diameter that weigh ≤ 29.5 kg (65 pounds)

The ASCS (Simple) Cart provides:

- 450 mm Cart
- 125mm (5 inches) Casters which are omni-directional, lockable casters, allowing for quick and easy transport of the system within hospitals and operating theatres.
- Shelf for supporting AtriCure ACM, ASU, and ASB products.
- Drawer for ancillary items.
- Tank holder and straps for securing tank(s).

SYSTEM DESCRIPTION

The Simple Cart supports medical equipment used in surgical theaters. The ASCS Cart supports the following AtriCure systems:

ATRICURE SYNERGY ABLATION SYSTEM

1. AtriCure® Ablation Sensing Unit (ASU) produces and delivers RF energy, in a bipolar mode, at a frequency of approximately 460 kHz, with a maximum output power ranging from 12 Watts up to 30 Watts depending on the operating mode.
2. AtriCure® Switch Matrix (ASB) is a reusable accessory interface module that allows simultaneous connection of AtriCure® Isolator Synergy™ Handpieces and Isolator™ Pens to the ASU RF generator.

ATRICURE CRYOSURGICAL SYSTEM

1. AtriCure® cryoICE BOX (ACM) is an electro-mechanical and pneumatic cryogenic surgical system that delivers a cryogenic energy source, namely Nitrous Oxide, to create lines of ablation through cardiac tissue.

INDICATIONS FOR USE

The AtriCure Simple Cart is intended to be used as an accessory to the AtriCure Synergy Ablation System and AtriCure Cryosurgical System.

No part of the AtriCure Simple Cart or its accessories is intended to make contact with the patient.

EQUIPMENT CLASSIFICATION

The AtriCure® Simple Cart is categorized as: Class 1 Equipment; Type CF Applied Part; IPX0.

ENVIRONMENTAL INDICATIONS

NOTE: Environmental Conditions:

- [1] Operational
 - Temperature: +10°C to +40°C (+50°F to +104°F)
 - Humidity: 30% to 75% relative humidity
 - Atmospheric Pressure: 70 kPa to 106 kPa (10.1 psi to 15.4 psi)
- [2] Storage and Transit
 - Temperature: -25°C to 70°C (-13°F to 158°F)
 - Humidity: 10% to 95% relative humidity
 - Atmospheric Pressure: 50 kPa to 120 kPa (7.2 psi to 17.4 psi)

PROPER USE

NOTE: The AtriCure® Simple Cart is ONLY intended for use with the equipment listed under the System Description.

NOTE: The cart should never be used to carry equipment other than what is specified in these instructions. The cart should not be operated on an incline exceeding 10°. During operation and storage, the casters should be locked securely.

NOTE: These instructions do not address specific instructions related to the equipment listed under System Description. The user should refer to product specific manuals provided with individual components.

Caution: Secure each tank with straps and ONLY use N2O Tanks ≤ 20 cm (8 inches) in diameter that weigh ≤ 29.5 kg (65 pounds)

Caution: Tanks must be secured prior to moving or transporting the cart.

EQUIPMENT PLACEMENT

1. Place cryoICE BOX (ACM) on Cart shelf. Refer to equipment manual if unit does not set level.
2. If applicable, stack ASU and ASB Units into rubber bumpers of each system to interlock units together.
3. Units should be placed in order as shown in FIGURE 1.
4. Place each tank on bottom support of Cart and nest into top and bottom tank bracket.
5. Secure each tank with strap on top bracket as shown in FIGURE 2.

CLEANING INSTRUCTIONS

NOTE: The Simple Cart Cleaning and Disinfection Instructions: (1) use a mild detergent (prepared to its specifications) and a damp cloth; and (2) do not use caustic, corrosive, or abrasive cleaning materials.

SERVICEABLE PARTS

NOTE: The Simple Cart does not have any serviceable parts. For servicing issues, contact AtriCure, Inc.

DISCLAIMER

Under no circumstances will AtriCure, Inc. be responsible for any incidental, special or consequential loss, damage, or expense, which is the result of the deliberate misuse of this product, including any loss, damage, or expense which is related to personal injury or damage to property.

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AtriCure Inc.