FIGURE 1

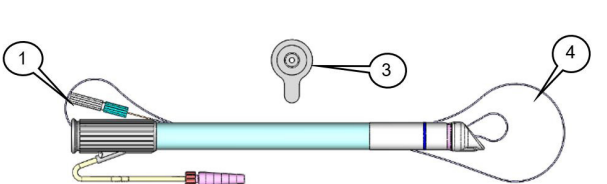
FIGURE 2


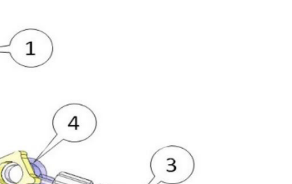
FIGURE 3

Table with 2 columns: Symptom, Action. Lists symptoms like 'Unable to advance instruments within canals' and corresponding actions like 'Lubricate canals with sterile water'.

FIGURE 4

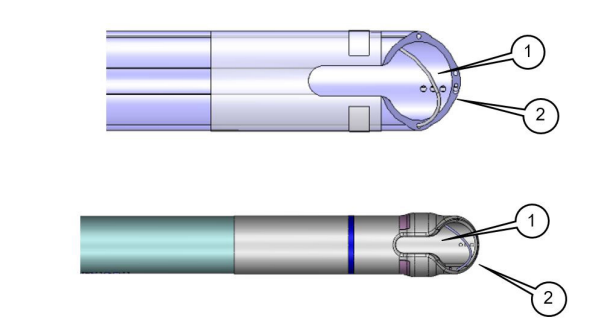PO13157 Rev D (US) restricts this device to sale by or on the order of a physician.

PRODUCT DESCRIPTION: The Canals with Guide (CCK-4130 & CCK-4140) is provided sterile, for single-use only. This product may be used in conjunction with the following devices produced separately:

INDICATIONS: The Canals with Guide is indicated for endoscopic use to provide cardiobascular surgical access for AtriCure instrumentation.

CONTRAINDICATIONS: None are known contraindications.

WARNINGS: The Canals with Guide is intended for single use only. Do not reprocess or reuse.

PRECAUTIONS: Endoscopic procedures should be performed only by physicians with adequate training with endoscopic techniques.

POTENTIAL COMPlications OF THE USE OF THE CANALS WITH GUIDE: (1) Tissue trauma to adjacent organs.

RECOMMENDED EQUIPMENT: 1. Vacuum tubing Set (CCK-1300).

CANAL SETUP: 1. Inspect all components. Check packaging to ensure there has been no product damage.

REQUIRED EQUIPMENT AND SUPPLIES PROVIDED BY HOSPITAL: 1. Vacuum tubing Set (CCK-1300).

CANAL DEPLOYMENT AND MANIPULATION: 1. Remove finger from the end of the guidewire and thread guidewire through available lumen.

CAUTION: Failure to replace torques correctly may result in permanent damage to the guidewire.

FIGURE 4. Canals with Guide in Guidewire Deployment Position: (1) Guidewire, (2) Distal End of Canals.

WARNING: Insertion or removal of the Canals with Guide white guidewire is intended as cause potential patient harm.

5. Before inserting instruments, lubricate canals lumen by injecting approximately 20 sterile water or saline through hole in the canals cap.

CAUTION: Do not manipulate canals by grasping the tab of the canals cap. Doing so may loosen or remove canals cap from canals casting rod functionality.

ATTENTION: Do not use the over-the-wire device. Remove the canals cap fully after the canals have fallen to the canals cap and the over-the-wire device.

TROUBLESHOOTING: 1. Unable to advance instruments within canals.

ABBREVIATIONS: - IFR: Instructions for Use

WARNANT INFORMATION: AtriCure warrants that reasonable care has been used in the design and manufacture of this instrument.

DISCLAIMER: The manufacturer is not responsible for any injuries or loss of product due to its use, including but not limited to, ensuring that the product is used as intended.

CAUTION: Federal Law (US) restricts this device to sale by or on the order of a physician.

PO13157 Rev D (US) restricts this device to sale by or on the order of a physician.

INDICATIONS: The Canals with Guide is indicated for endoscopic use to provide cardiobascular surgical access for AtriCure instrumentation.

CONTRAINDICATIONS: None are known contraindications.

WARNINGS: The Canals with Guide is intended for single use only. Do not reprocess or reuse.

PRECAUTIONS: Endoscopic procedures should be performed only by physicians with adequate training with endoscopic techniques.

POTENTIAL COMPlications OF THE USE OF THE CANALS WITH GUIDE: (1) Tissue trauma to adjacent organs.

RECOMMENDED EQUIPMENT: 1. Vacuum tubing Set (CCK-1300).

CANAL SETUP: 1. Inspect all components. Check packaging to ensure there has been no product damage.

REQUIRED EQUIPMENT AND SUPPLIES PROVIDED BY HOSPITAL: 1. Vacuum tubing Set (CCK-1300).

CANAL DEPLOYMENT AND MANIPULATION: 1. Remove finger from the end of the guidewire and thread guidewire through available lumen.

CAUTION: Failure to replace torques correctly may result in permanent damage to the guidewire.

FIGURE 4. Canals with Guide in Guidewire Deployment Position: (1) Guidewire, (2) Distal End of Canals.

WARNING: Insertion or removal of the Canals with Guide white guidewire is intended as cause potential patient harm.

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